# Rhinodoron®

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### Composition :

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## nasal spray with Aloe Vera

### Ingredients:

1 g of isotonic solution contains:  
Sodium chloride 5 mg,  
Potassium chloride 5 mg,  
Aloe vera gel (Aloe barbadensis leaf in juice) 5 mg,  
Water for injections.

### What is Rhinodoron and when can it be used?

To ensure easy breathing, the nasal cavity must be completely clear and slightly moist.

Rhinodoron® contains Aloe vera gel in an isotonic salt solution. This means that the salt content of the product is adjusted to match that of the body's fluids, so the solution is well tolerated by the mucous membranes. Rhinodoron® is ideal to clear and moisturise the nasal passages, particularly in the case of dry nasal mucosa or build up of dry crusts, in the dry air of homes or offices or in the winter and hayfever seasons. Rhinodoron® is ideal for the hygienic care of the nasal passages of infants who cannot blow their own noses.

Rhinodoron®, thanks to its unique spray device prevents nasal secretions and bacterial contamination from entering the bottle.

For technical reasons the bottle cannot be completely emptied. The bottle is overfilled to cover the residual volume.

### How do I use Rhinodoron®?

2-6 times daily, 1-2 sprays into each nostril.

### Directions for use:

Do not take orally.

To use, follow these instructions:

Shake before use.

Remove the protective cap.

Hold the spray vertically and press the spray top until a fine mist appears (see picture 1).

### Adults and children:

Clean the nose carefully and thoroughly.

Lean the head to one side and insert the spray cap into the upper nostril (see picture 2).

Repeat the procedure for the other nostril.

### For babies (3 month and above):

Clean the nose carefully, if necessary with the help of a cotton bud.

Lay the child down with the head to one side.

Spray the solution into the upper nostril.

Repeat the procedure for the other nostril.

Wipe the spray cap after every use.

### Caution

Due to the pressure of the spray, care should be taken when used on babies and small children.

### When should Rhinodoron® not be used?

After nasal operations or accidents involving the nose or if sensitive to Aloe vera.

### Can Rhinodoron® be used during pregnancy and feeding?

Rhinodoron® can be used during pregnancy and breast feeding.

Store at room temperature.

KEEP OUT OF THE REACH AND SIGHT OF CHILDREN.

### Are there any side effects with Rhinodoron®?

None known.

### Where can I buy Rhinodoron® and in which pack sizes?

Available in pharmacies independent health food stores and other retail outlets without a doctor's prescription.

20 ml.